
Two-Piece Pouching System

Collect Supplies

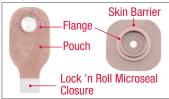

Gather pouch, skin barrier, disposal bag, tissue and/or cloth for cleaning, and water. Optional: scissors, pen, measuring guide, and/or accessories.

Prepare the Skin Barrier and Pouch



Measure the stoma using a stoma measuring guide before every barrier application (if using a pre-sized barrier opening, go to step 7).

Add Paste or Rings If Needed



If appropriate, apply a barrier ring or thin bead of paste around the opening following the instructions for use.



Remove Used **Pouching System**



Carefully lift an edge of the adhesive and peel downward, gently pushing the skin away from the skin barrier. Properly dispose of used pouch and skin barrier. Do not flush down toilet.

Clean and Dry Skin



Clean the skin with water or with gentle soap as recommended by your healthcare professional. Rinse and dry thoroughly. Do not use oils, powder, lotion, or lotiontype soaps on the skin.

6

Using small curved scissors, carefully cut an opening in the skin barrier to match the pattern. Do not cut beyond the line on the release liner.

Check Condition



3 Check to make sure the skin around your stoma (peristomal)

*Contact your WOC/NSWOCC nurse if you notice irritation, rash or redness

or redness.*

is intact without irritation, rash



Remove the release liner from the skin barrier.

Apply the New **Pouching System**



Trace the determined barrier size

opening onto the skin barrier. The

skin barrier should fit where the

skin and stoma meet. Verify that

no skin is showing between the

skin barrier and the stoma to help prevent leakage and skin irritation.

Center the skin barrier opening over the stoma in a diamond shape. For tape bordered barriers, remove the backing paper on both sides and press adhesive against the skin.

Close bottom of pouch. If using a drainable pouch, close the Lock 'n Roll microseal closure at the bottom of the pouch or apply a pouch clamp. Refer to the Lock 'n Roll Microseal Closure Care Tip for more information on how to use this closure system.




Apply gentle pressure to the skin barrier for about a minute for best adhesion.



Attach the pouch to the skin barrier by engaging the lower edges of the pouch flange and the skin barrier flange.

Press the two flanges together. A series of "clicks" will confirm you are doing it correctly. No more "clicks" means the pouch is secured.

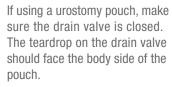

If desired, apply the cap provided in the box.



Two-Piece Pouching System

LIST OF PRODUCTS **TIPS** STOCK NO • The skin around your stoma should not show any **Two-Piece Pouching System** signs of rash or sores Skin barrier Irritated skin should be evaluated by a healthcare Pouch professional **Accessories** • If using a barrier ring or paste, apply it to the sticky side of the skin barrier next to the stoma opening Barrier rings as directed Ostomy belt · Your stoma will probably be swollen after surgery. It may take several weeks or months for the stoma to Paste shrink to its permanent size Lubricating deodorant • The stoma may bleed slightly when you clean it Stoma powder The use of skin protective wipes are not recommended under extended wear skin barriers Other__ as they may decrease wear time Additional Instructions

For product questions, sampling needs, or detailed clinical questions concerning our products in the US, call 1.888.808.7456. In Canada call 1.800.263.7400.

Skin should be clean and dry before applying the skin barrier.

Prior to use, be sure to read the Instructions for Use for information regarding Intended Use, Contraindications, Warnings, Precautions, and Instructions.

The Hollister logo, Lock 'n Roll and "Healthy skin. Positive outcomes." are trademarks of Hollister Incorporated. © 2018 Hollister Incorporated. 923081-1118 US-00666

Hollister Incorporated 2000 Hollister Drive Libertyville, Illinois 60048 1.800.323.4060

www.hollister.com

Hollister Limited 95 Mary Street Aurora, Ontario L4G 1G3 1.800.263.7400

