

Patient: 42-year-old female presented with a right groin open post-surgical wound

Wound characteristics and prior treatment:

- Post-surgical wound was from surgical excision
- Wound measurement: 3.0cm x 3.0cm x 2.0cm deep
- No pain management was required

Prior treatment:

- Wound treated daily with an enzymatic debrider and gauze dressing for 14 days with minimal to no improvement noted
- The physician and patient agreed to introduce the addition of an Endoform dermal template to assist in wound healing

Treatment:

- Week 1: Sharp debridement was performed in the office. Endoform dermal template was placed into the wound and covered with a secondary foam dressing
- Week 2: Patient returned with Endoform dermal template still partially visible within the wound upon secondary dressing removal. Another Endoform dermal template was trimmed and placed into the wound and covered with a secondary foam dressing. Wound measurements were 2.5cm x 2.0cm x 1.5cm
- Week 3: Improved granulation tissue was noted, the wound was shallower, with no Endoform dermal template dressing residue visible in the wound bed
- Week 4: Wound ready for delayed primary closure

Results:

- Patient reported no pain throughout the treatment with Endoform dermal template
- Treatment goal was met with use of Endoform dermal template



Presentation: Wound measurement
3.0cm x 3.0cm x 2.0cm
Endoform dermal template and secondary
foam dressing were applied



Week 2: Wound measurement
2.5cm x 2.0cm x 1.5cm
A second application of Endoform dermal
template covered by a secondary foam
dressing was applied



Week 4: Achieved in-office closure



Manufactured for
Hollister Wound Care
2000 Hollister Drive
Libertyville, IL 60048
1.888.740.8999

Distributed in Canada by
Hollister Limited
95 Mary Street
Aurora, Ontario L4G 1G3
1.800.263.7400

www.hollisterwoundcare.com

Case provided by:
Paul M. Glat, MD, FACS; 191 Presidential Blvd. Suite 101, Bala Cynwyd, PA

CAUTION: Federal (USA) Law restricts this device to sale by or on the order of a physician.
Prior to use, be sure to read the entire Instructions for Use package insert supplied with the product.
922216-0213

Hollister and Logo is a trademark of Hollister Incorporated.
Endoform is a trademark of Mesynthes Limited.
©2013 Hollister Incorporated.