

Endoform™ Dermal Template

WOUND CARE DRESSING
APPLICATION GUIDE



**The strength of a
dermal template.**

**The simplicity of
collagen.**

1 Prepare



Cleanse the wound bed.

2 Apply



Trim dressing to fit the wound.

If needed, multiple sheets can be used to cover the surface of the wound.

Place the dry dressing in the wound and hydrate with exudate or saline solution until moistened.

3 Protect



The Endoform dermal template may need protection from adhering to the secondary dressing.

Cover the Endoform dermal template with a non-adherent dressing.

Example: Restore Contact Layer FLEX dressing.

4 Cover



Select appropriate secondary dressing based on type of wound and amount of exudate.

If little to no exudate, apply a hydrogel to maintain appropriate moisture balance.

Example: Hydrofera Blue dressing (shown) or Restore border foam dressing with Triact technology.

5 Secure



If needed, secure cover dressing with rolled gauze.

The Endoform dermal template may be used in conjunction with compression therapy and negative pressure wound therapy.

6 Reapply



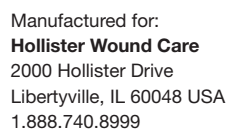
Reapply the Endoform dermal template every 5-7 days or as needed.

It is not necessary to remove any residual Endoform dermal template during dressing changes or reapplication of product.

WOUND CARE DRESSING APPLICATION GUIDE

Stock No	Product Size	Quantity/Box	HPCPS
529311	2" x 2" (5 cm x 5 cm)	10	A6021
529312	2" x 2" (5 cm x 5 cm) fenestrated	10	A6021
529313	4" x 5" (10 cm x 12.7 cm)	10	A6022
529314	4" x 5" (10 cm x 12.7 cm) fenestrated	10	A6022

Additional Instructions

[illegible]

Distributed in Canada by:
Hollister Limited
95 Mary Street
Aurora, Ontario L4G 1G3
1.800.263.7400

For product questions, sampling needs, or detailed clinical questions concerning our products in the US call **1.888.740.8999**. In Canada call **1.800.263.7400**.

HPCPS are for reference only and subject to change
Hollister and logo is a trademark of Hollister Incorporated.
Endoform is a trademark of Mesynthes Limited.
©2013 Hollister Incorporated.
922350-914